We believe

“In medical devices of the highest quality, approved by the strictest international standards and by surgeons all over the world who trust us, that’s why we innovate...”

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Who we are | Quienes somos

IFSA has over 40 years experience in manufacturing and sales of medical devices. The company has sustained a continuous growth in the Medical Devices field. Our sutures brand, Atramat®, is sold and recognized in over 70 countries around the World, enjoying market and surgeon acceptance based on our high quality standards, growing range of products, as well as fair and competitive pricing.

We maintain our quality standards based on the most demanding international regulations, thus all of our products have the European CE Mark, the US 510K, and our plants are continuously verified and certified by FDA and notify bodies to review the compliance with ISO-9001 and ISO-13485. This exemplifies our utmost responsibility level with health care, patients, surgeons and nurses worldwide.

Thanks to our continuous research and development programs, we successfully offer innovative products in balance with modern medicine techniques and concerned to improve patients life and treatment quality.

IFSA recognizes that the difference between companies lies with their people. This is why we promote a socially responsible culture within our staff, in order to become ambassadors of our values.

Our mission follows these values. Therefore in the future and based on our quest to maintain quality, innovation and integrity, we will continue to be a dynamic company, living with a common purpose: Providing quality products and services for the medical community including patients, nurses and surgeons, while maintaining an environment of conscious behaviour.

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Our Mission:

To develop and commercialize reliable and innovative medical devices for a better healthcare.

Our Vision for 2017 is:

An important part of our income should come from new lines of business.

Substantially increase our international presence in our closure technology products

Our product line should be the first choice among our customers

To continue being a financially viable company

Atramat shall remain a company that maximizes the potential of their personnel

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Quality in everything we do is the top priority in Atramat,[®] that is why:

- Atramat checks 100% of the products within the production stage
- All our products comply with the US Pharmacopeia norms and all other quality certificates
 - Constant study and research on how to make our products better
 - High quality standards in our production lines and procedures

Our plants have the following quality certifications:

ISO 13485:2003
 ISO 9001:2008
 GMP's Certificates (COFEPRIS and ANVISA),
 ISO 13485:2003 issued by CMDCAS
 SABS (RSA)
 CE Mark
 US FDA 510K

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ISO 13485:2003
 ISO 9001:2008
 GMP's Certificates (COFEPRIS and ANVISA),
 ISO 13485:2003 issued by CMDCAS
 SABS (RSA)
 CE Mark
 US FDA 510K

ISO 13485:2003

We believe

“that patients deserve the best quality treatments and only the best quality in products, that’s why we innovate...”

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Our Needles | Nuestras Agujas

Depending on the sharpening technique employed, our needles can be classified as follows:

- Standar
- Premium
- 300plus
- Supra-sharp
- AtraGlide®

Our needles can be described according to the following characteristics:

- Curvature
- Length
- Wire Diameter
- Type of needle tips

In the pages that follow you will find our extensive catalogue of needles. Any specific needle which is not illustrated can be individually requested.

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Straight Half Curve / Recta Media Curva	Straight / Recta

Bi-Curve / Bi-Curva	J Needle / Aguja J

AtraGlide® needles: sharper and smother

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Our newest needle AtraGlide® has the best design and technology, standing out through its performance.

Its refined tip cutting edges are remarkably sharp and smooth, thus facilitating penetration with great control.

AtraGlide® tip is very narrow at the needlepoint, maximizing penetration as it moves through the body tissue.

Its cutting edges expand outwards and become wider in order to facilitate penetration, further along the cutting edges retract inwards and eventually integrate seamlessly into the body of the needle in order to decrease tissue damage.

AtraGlide® has a special coating, which contributes on easing the passage through tissue.

The AtraGlide® needle is composed of a superior quality alloy and is an excellent alternative for Plastic Surgery, Cosmetic and Reconstructive.

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9

Atramat® has proudly developed a new generation of needles which perfectly combine a special stainless 300 plus steel series alloy, with optimum geometric design, and a technologically innovative coating, resulting in an ideal harmonic needle.

Alloy

The technological development adopted for our new line of precision surgical needles is subject to the optimal selection of steel alloy based on stringent criteria. This is to ensure that the end product is of superior quality and suitably designed for the most complex specialty surgery.

Geometry

The exceptional geometric design of both the body and tip of the needle, means that it offers surgeons both outstanding strength while penetrating tissue and precise control.

Coating

A special thermal coating results in superior and easier penetration of tissue, coupled with minimal resistance and smoothness. This in turn reduces the effect of dragging, and means that the needle point remains sharper for a longer duration.

- Proven effectiveness.
- Technologically innovative.

300plus Anti-Reflective Dark Needle

Besides of the remarkable characteristics of the 300plus Needle in its alloy, geometry and coating, we have added a dark non-reflective finish for an optimum contrast in the surgical field, avoiding light reflection and enhancing the visualization. This is very convenient in micro-vascular surgery.

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
Our Needles | Nuestras Agujas

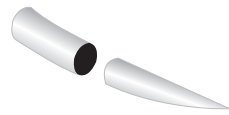TAPER POINT AHUSADA


This tip profile is designed to provide easy penetration of intended tissue. It is widely used in tissue with moderate ease of penetration, such as the peritoneum-fascia muscle, fatty tissue and intestine. It is also widely used in veins and arteries.

Specialities include: General Surgery, Gastro-intestinal, Cardiovascular and Gynaecology.

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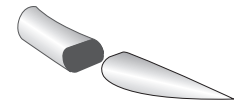
The tapered cross section immediately behind the tip has been flattened to an oval shape in our round bodied needles; facilitating the penetration through tissue layers. It is widely used in General and Gastro-intestinal Surgery.

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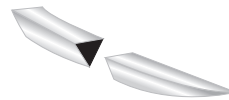REVERSE CUTTING REVERSO CORTANTE


Cutting needle characterised by its triangular cutting pattern, with the apex cutting edge on the outside of the needle curvature. Widely used in Reconstructive Plastic Surgery and cuticular closing.

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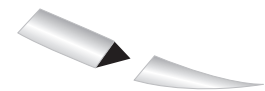CONVENTIONAL CUTTING CORTANTE CONVENCIONAL


Cutting needle characterised by its triangular cutting pattern, with the apex cutting edge on the inside of the needle. Widely used in Reconstructive Plastic Surgery and in cuticular closing.

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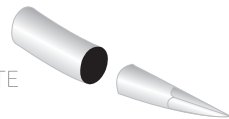TAPER CUTTING CUERPO REDONDO PUNTA CORTANTE

Widely used for tissue with tough penetration that requires a circular pattern. This avoids damage to the tissue from the tension caused by the suture. It is also used for tendons, Orthopaedics and sternum closure, due to its great penetration and strength.

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PREMIUM SPATULA ESPATULADA PREMIUM

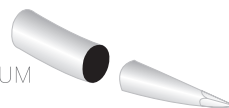
Due to its thin, flat, knife-like shape, and lateral, bottom cutting edges, passing through tissue is very smooth thus minimizing trauma and avoiding accidental penetration of deeper tissue. Recommended for Ophthalmic procedures. (For example Cataract and Front Segment Surgery).

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PREMIUM TAPER CUTTING AHUSADA PUNTA CORTANTE PREMIUM


Round bodied needle with a special precision cutting point, allows easier penetration through calcified tissue. The needle is manufactured from a special stainless steel alloy that provides excellent tissue penetration and high resistance to bending. Ideal for placing heart valves in Cardiovascular Surgery.

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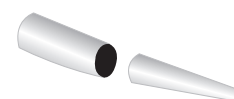PREMIUM BLUNT POINT ROMA PREMIUM

Cylindrical bodied needle with a rounded point (Rome). Particularly designed for suturing soft tissue such as in liver and kidneys. This needle has a special coating that eases penetration through tissue.

Due to its tip profile, it eliminates accidental glove puncture; while remaining sharp enough to penetrate fascia and muscle.

Note: This needle should not be used for skin.

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Our Needles | Nuestras Agujas

SUPRA-SHARP REVERSE CUTTING REVERSO CORTANTE SUPRA-SHARP

This new reverse cutting needle is doubly sharpened with a triangular cutting pattern. The improved geometry of the needle reduces the cutting edges angle, improving penetration and control through the tissue. Frequently used in reconstructive Plastic Surgery.

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SUPRA-SHARP CONVENTIONAL CUTTING CORTANTE CONVENCIONAL SUPRA-SHARP

This new conventional needle is doubly sharpened with a triangular cutting pattern. The improved geometry of the needle reduces the angle of the cutting edges, improving the penetration and control through the tissue. Frequently used in reconstructive Plastic Surgery.

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300plus TAPER POINT AHUSADA 300plus

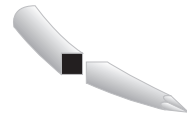
A taper point needle which has a cylindrical body starting at the point of assembly. The middle portion of the needle presents a special square section; designed to give superior control when using with needle holders as well as enhanced physical strength. From the proximal portion to the edge it is characterised by a conic shape (with a gradual slimming) until the needle point. The "SB" needle has a special coating that favours penetration through tissue. Usos: General Surgery, Gastroenterology, Gynaecology and Obstetrics, Cardiovascular and Neurosurgery.

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300plus TAPER CUTTING AHUSADA PUNTA CORTANTE 300plus


Cylindrical bodied needle with high precision cutting edge. The middle portion of the needle presents a special square section; it is designed to provide superior control when used in conjunction with needle holders as well as a physically superior strength. The proximal portion to the needle point has three triangular cutting edges, specifically designed for piercing through calcified tissue and atheromatous plaques with great ease. The "YB" needle has a special coating that enhances penetration through tissue. Usos: Cardiovascular and Micro-vascular surgery.

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300plus TAPER POINT AHUSADA 300plus

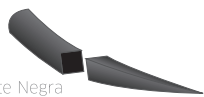Non-Reflective Dark Needle / Aguja Antirreflejante Negra


A taper point needle which has a cylindrical body starting at the point of assembly. The middle portion of the needle presents a special square section; designed to give superior control when it is used with needle holders as well as enhanced physical strength. From the proximal portion to the edge, it is characterized by a conic shape (with a gradual slimming) until the needle point. The "SV" needle has a special coating that favors penetration through the tissue. The dark surface of the needle allows its easy visualization in the surgical theatre by avoiding any reflection. Very suitable in micro-vascular surgery. Usos: Cardiovascular and Micro-vascular Surgery.

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300plus TAPER CUTTING AHUSADA PUNTA CORTANTE 300plus

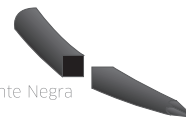Non-Reflective Dark Needle / Aguja Antirreflejante Negra

Cylindrical bodied needle with a high precision cutting edge. The middle portion of the needle presents a square section; designed to provide superior control when used in conjunction with needle holders as well as a physically superior strength. The proximal portion to the needle point has three triangular cutting edges, specifically designed for piercing through calcified tissue and atheromatous plaques with great ease. The "YV" needle has a special coating that enhances penetration through the tissue. The dark surface of the needle allows improved visualization in the surgical theatre by avoiding any reflection. Very suitable for Micro-vascular Surgery. Usos: Cardiovascular Surgery.

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ATRAGLIDE® CUTTING CORTANTE ATRAGLIDE®

This needle is an evolution in geometric design.

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It has four cutting edges, micro sharpened with a modified diamond shaped tip. This remarkably sharp, ultra smooth, four-edged tip is very narrow at the needle point. As we move towards the body of the needle these cutting edges expand outwards and become wider to maximize penetration. Further along the cutting edges retract inwards and eventually integrate seamlessly into the body of the needle.

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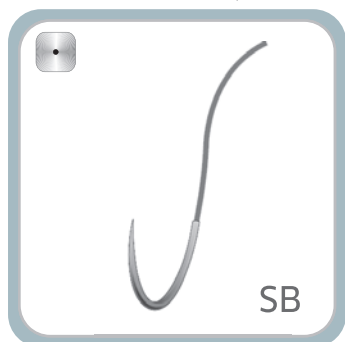
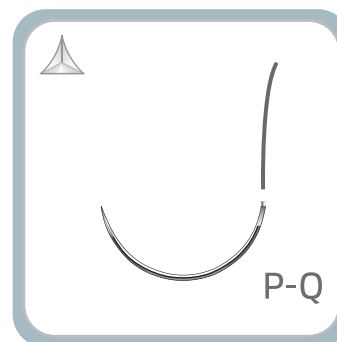
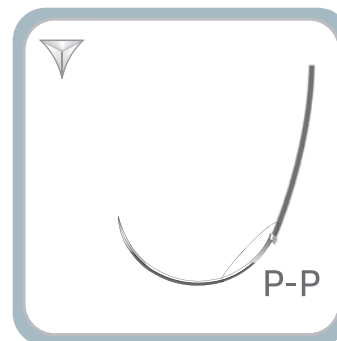
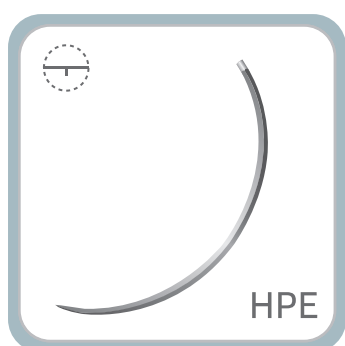
AtraGlide®

The AtraGlide® needle is constructed with a superior quality alloy
and is an excellent alternative in Plastic Surgery.
Superior ease to the penetration.
Extraordinary Geometry.
Cosmetic results.

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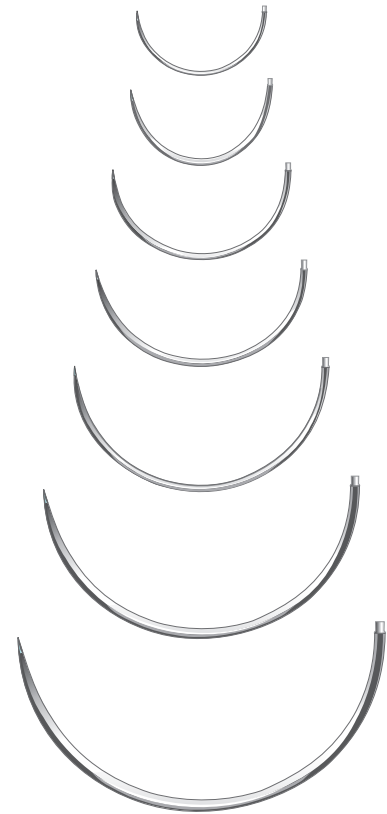
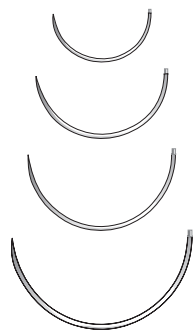
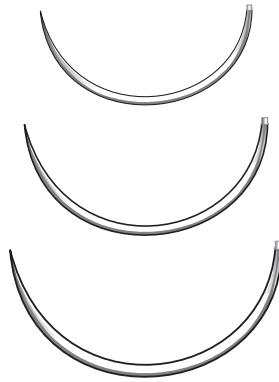
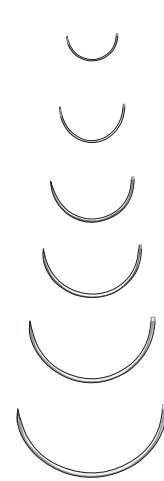
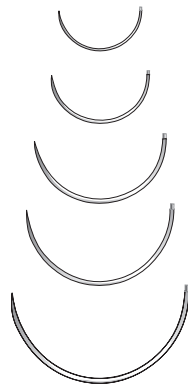
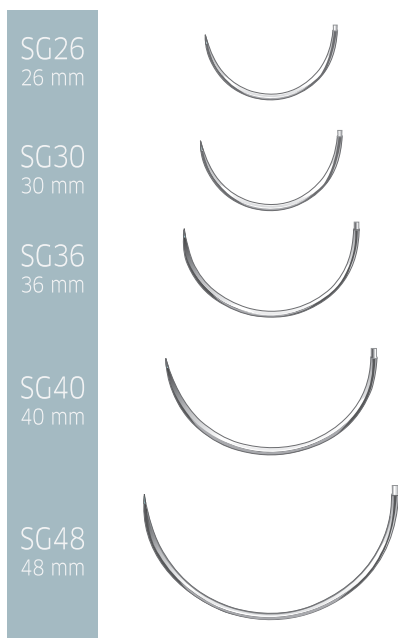
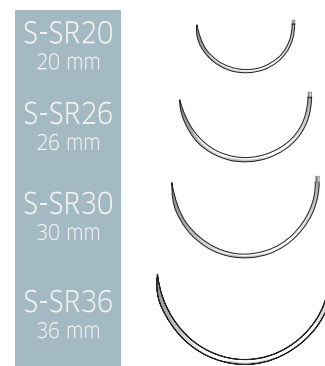
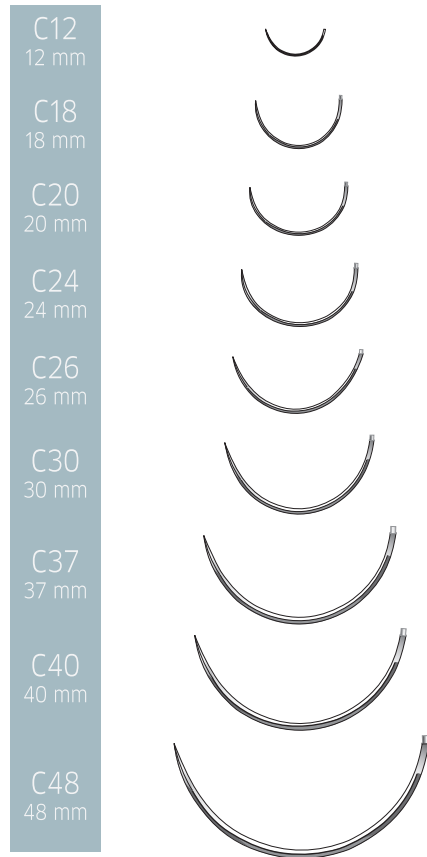
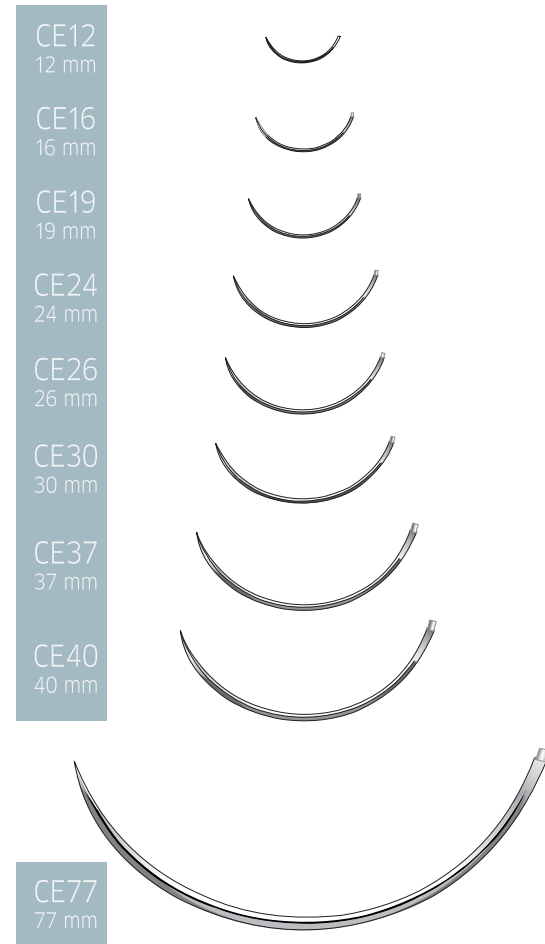
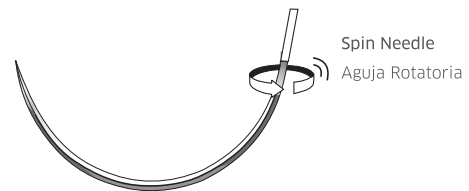
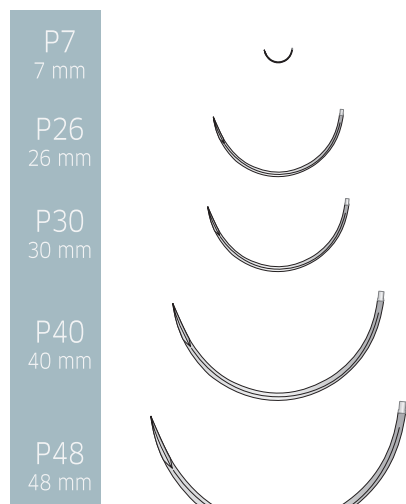
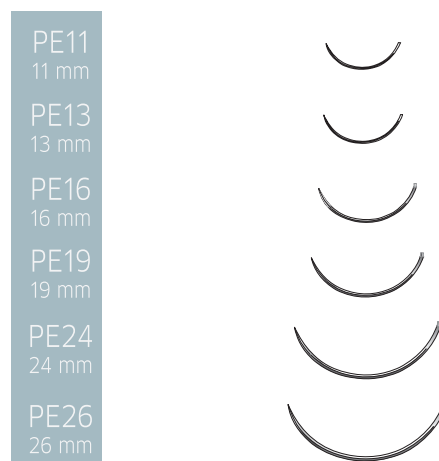
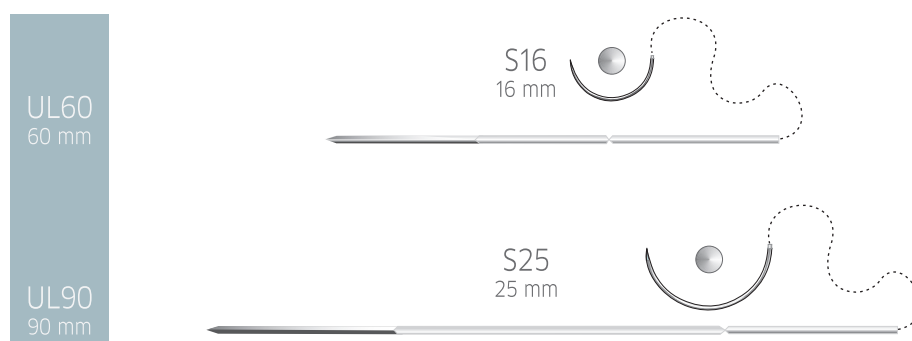
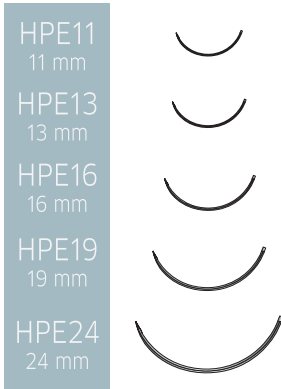
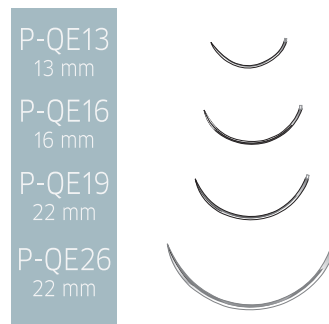
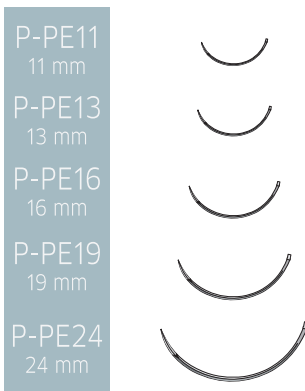
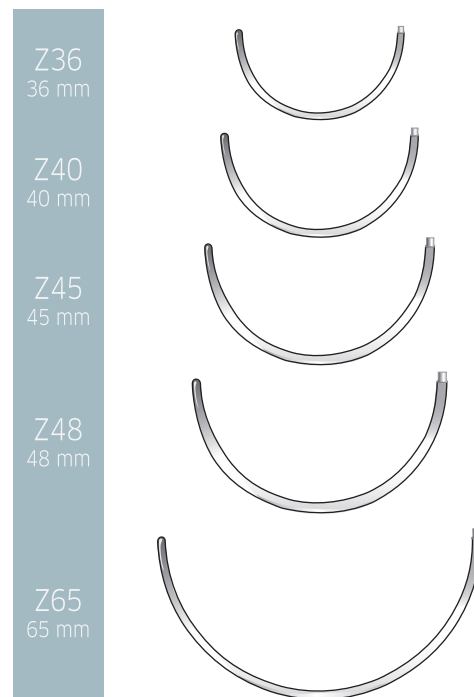
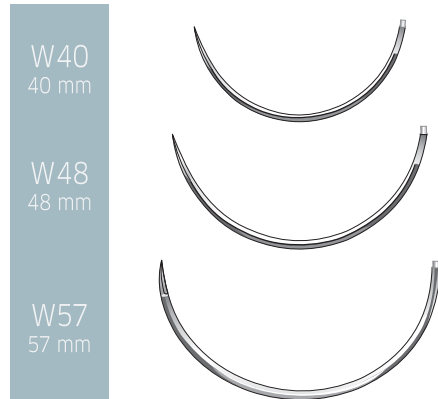
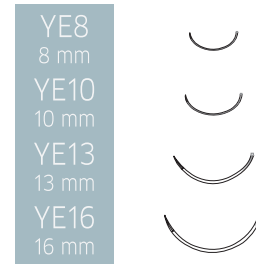
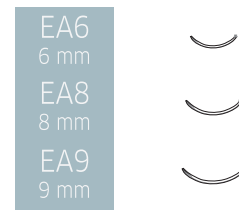
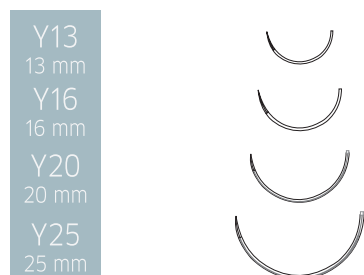
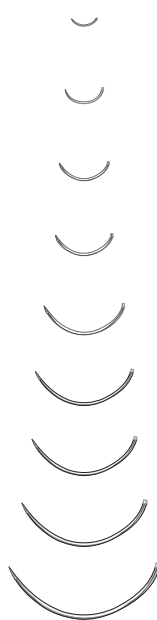
24

The examples of bar coding shown below explain the information contained in the code and their meaning.

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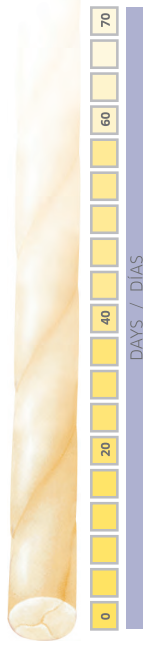
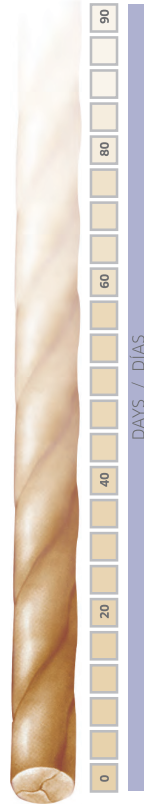
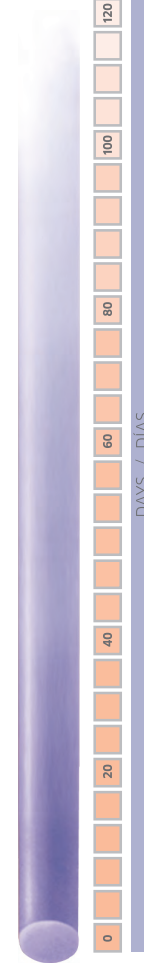
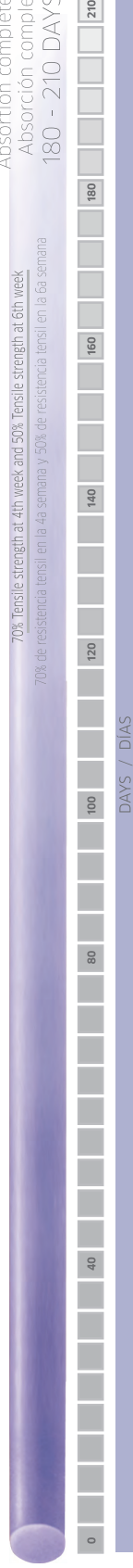
Absortion complete between
Absorción completa entre
180 - 210 DAYS / DÍAS

We believe

“In our work; and all the effort, attention and quality that we apply to each and every of the medical devices we produce, that’s why we innovate...”

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A blue suture thread is shown against a white background. The thread starts from the left edge, forms a series of wavy loops, and ends on the right with a silver-colored needle. The text "Our Sutures | Nuestras Suturas" is centered in the middle of the image.

Our Sutures | Nuestras Suturas

Synthetic, absorbable, monofilament suture, presented in violet or undyed.

P-dioxanone polymer.

Offers minimal tissue reactivity.

Absorption occurs through slow hydrolytic action completed in approximately 180- 200 days.

The material retains approximately 70% of its tensile strength by the 4th week, and 50% by the 6th week.

Color code: Silver.

Frequently used in tissue coaptation requiring a longer healing period.

CHARACTERISTICS + BENEFITS

High tensile strength = Excellent tissue support.

Absorption through hydrolysis = Predictable absorption rate and minimal tissue reaction.

Excellent knot coupling = Provide excellent security.

Hermetically sealed packaging = Ensured stability and sterility of the product.

ABOUT

Recommended for any surgical procedures requiring a single-thread, synthetic, absorbable suture.

KNOTTING: When used with thick gauges (2, 1, 0, 2-0) it is preferable to tie both a surgeon knot and simple knot. This will allow the surgeon to adjust the tension accordingly. Once adjusted, an additional square knot is recommended according to the surgical circumstances and the surgeon experience.

When applied in thin gauges (3-0, 4-0, 5-0, 6-0, 7-0), initially two square knots are recommended with 2 or 3 additional bowknots, leaving the ends of the knots 4 to 5 mm long subject to the surgical circumstances and the surgeon experience.

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Made of a copolymer of Poly(glycolide co-epsilon-caprolactone).

The tissue reactivity is minimal.

Absorption occurs through progressive hydrolytic action; completed between 90 and 120 days.

The material retains approximately 60% of its initial tensile strength by the first week.

Color code: Light Orange.

Frequently used in soft tissue coaptation.

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Monofilament with uniform diameter = Smooth passage through the tissue.

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Violet color = Excellent visibility in the operating theatre.

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ABOUT

Indicated for general approximation of soft tissue.

KNOTTING: When used with thick gauges (1, 0, 2-0) it is preferable to tie both a surgeon knot and simple knot. This will allow the surgeon to adjust the tension accordingly. An additional square knot is recommended according to the surgical circumstances and the surgeon experience.

When applied in thin gauges (3-0, 4-0, 5-0, 6-0), initially two square knots are recommended with 2 or 3 additional throws, leaving the ends of the knots 4 to 5 mm long subject to the surgical circumstances and the surgeon experience. Damaging the suture thread with forceps, needle holders and so on, should be avoided.

The ATRAMAT® PGC25 is a copolymer composed of Poly (glycolide-co-Epsilon-caprolactone) in the following approximate percentages:

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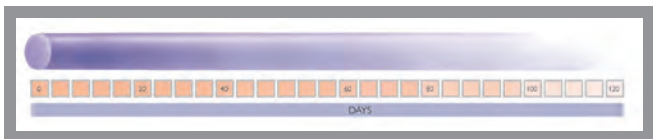
Policaprolactona.....25%

Suturas Absorbibles
Atramat®PGC25

Retains 60% (violet thread) 50% (non-dyed thread) tensile strength at 1st week and 30% (Violet thread) 20% (non-dyed thread) tensile strength at 2nd week

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Absorbable Sutures

Atramat®PGA

Synthetic, absorbable, multifilament braided suture, presented in violet or undyed.

Composed of a Polyglycolic acid coated with polycaprolactone and calcium stearate.

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Absorption occurs through progressive hydrolytic action, completed between 60 and 90 days.

The material retains approximately 70% of its initial tensile strength by the end of two weeks.

Color code: Violet.

Used in soft tissues coaptation, ligatures, and a wide variety of surgical specialties.

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ABOUT

When implanted in the tissue, the ATRAMAT® Polyglycolic Acid has minimal tissue reaction on a microscopic basis. This is comparable to other absorbable synthetic sutures of Polyglycolic Acid. However, ATRAMAT® Polyglycolic Acid, offers a predictable absorption rate and high tensile strength over and above the USP standard, thus providing vital artificial support to the tissue during the critical healing period. The absorption occurs through progressive hydrolytic action completed in approximately 60 to 90 days.

The ATRAMAT® Polyglycolic Acid is coated with Polycaprolactone and Calcium Stearate in the following approximate percentages:

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Regular Taper Point
Ahusada Regular

1/2 Circle
Círculo 

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 6-0 EP 0.7	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3
		70 cm	R2096	R2095			
		75 cm		R2095-75	R2094-75	R2093-75	R2092-75
R20 20 mm		90 cm				R2093-90	

Regular Taper Point
Ahusada Regular

1/2 Circle
Círculo 

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4
		70 cm	R2695	R2694	R2693	R2692		
		75 cm		R2694-75	R2693-75	R2692-75	R2691-75	R2698-75
R26 26 mm		90 cm			R2693-90	R2692-90		
		75 cm			R3093-75	R3092-75	R3091-75	
R30 30 mm		90 cm				R3092-90		
R37 37 mm		75 cm			R3793-75	R3792-75		

Heavy Taper Point
Ahusada Gruesa

1/2 Circle
Círculo 

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4	USP 2 EP 5
G26 26 mm		70 cm	G2693	G2692	G2691		
		70 cm <small>Undyed/Incoloro</small>		G2692-B			
		75 cm	G2693-75	G2692-75	G2691-75	G2698-75	G2699-75
		90 cm	G2693-90	G2692-90	G2691-90	G2698-90	
G30 30 mm		75 cm	G3093-75	G3092-75	G3091-75	G3098-75	G3099-75
		90 cm		G3092-90	G3091-90		
G37 37 mm		70 cm		G3792	G3791	G3798	
		75 cm	G3793-75	G3792-75	G3791-75	G3798-75	G3799-75
		90 cm		G3792-90	G3791-90	G3798-90	G3799-90
G40 40 mm		75 cm	G4093-75	G4092-75	G4091-75	G4098-75	G4099-75
		90 cm		G4092-90	G4091-90	G4098-90	G4099-90

Heavy Taper Point
Ahusada Gruesa

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4	USP 2 EP 5
G48 48 mm		75 cm	G4892-75	G4891-75	G4898-75	G4899-75
		90 cm	G4892-90	G4891-90	G4898-90	G4899-90
G65 65 mm		75 cm				G6599-75
		90 cm				G6599-90
G76 76 mm		75 cm	G7692-75			
G76 76 mm		90 cm		G7691-90		

Heavy Taper Point
Ahusada Gruesa

5/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4
GH26 26 mm		70 cm	GH2692	GH2691	GH2698

Heavy Taper Point
Ahusada Gruesa

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 1 EP 4
GE50 50 mm		75 cm	GE5098-75

Premium Regular Taper Point
Ahusada Regular Premium

1/2 Circle
círculo

Thread Gauge
Calibre de la Hebra

Needle Aguja	Length Longitud	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5
	75 cm	SR1795-75	SR1794-75			
	75 cm		SR2094-75	SR2093-75		
	75 cm		SR2694-75	SR2693-75	SR2692-75	
	75 cm				SR3092-75	SR3091-75
	75 cm				SR3692-75	SR3691-75

Premium Heavy Taper Point
Ahusada Gruesa Premium

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Needle Aguja	Length Longitud	USP 0 EP 3.5	USP 1 EP 4	USP 2 EP 5
	75 cm	SG3691-75	SG3698-75	
	75 cm		SG4098-75	
	90 cm	SG4091-90	SG4098-90	SG4099-90
	75 cm			SG4899-75
	90 cm		SG4898-90	

Reverse Cutting
Reverso Cortante

1/2 Circle
Círculo 

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4	USP 2 EP 5
C26 26 mm		75 cm	C2693-75	C2692-75	C2691-75		
C30 30 mm		75 cm		C3092-75	C3091-75		
C37 37 mm		75 cm	C3793-75	C3792-75	C3791-75	C3798-75	C3799-75
		90 cm	C3793-90	C3792-90	C3791-90	C3798-90	
C40 40 mm		75 cm			C4091-75	C4098-75	C4099-75
		90 cm		C4092-90	C4091-90	C4098-90	C4099-90
C48 48 mm		75 cm				C4898-75	C4899-75
		90 cm				C4898-90	C4899-90

Reverse Cutting
Reverso Cortante

3/8 Circle
Círculo 

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2
CE12 12 mm		45 cm	CE1295-45	CE1294-45	
		45 cm Undyed/Incoloro	CE1295-45B		
CE16 16 mm		45 cm		CE1694-45	CE1693-45
		75 cm		CE1694-75	
		75 cm Undyed/Incoloro	CE1695-75B	CE1694-75B	

Reverse Cutting
Reverso Cortante

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 6-0 EP 0.7	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2
CE19 19 mm		45 cm			CE1994-45	CE1993-45
		45 cm <small>Undyed/Incoloro</small>			CE1994-45B	
		70 cm			CE1994	
		70 cm <small>Undyed/Incoloro</small>	CE1996-B	CE1995-B		
		75 cm		CE1995-75	CE1994-75	CE1993-75
		75 cm <small>Undyed/Incoloro</small>	CE1996-75B	CE1995-75B	CE1994-75B	CE1993-75B

Reverse Cutting
Reverso Cortante

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4
CE24 24 mm		75 cm	CE2494-75	CE2493-75	CE2492-75	CE2491-75	
		75 cm <small>Undyed/Incoloro</small>		CE2493-75B	CE2492-75B		
CE30 30 mm		75 cm		CE3093-75	CE3092-75	CE3091-75	
CE37 37 mm		75 cm			CE3792-75		CE3798-75

Straight Cutting
Recta Cortante

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3
U60 60 mm		75 cm			U6092
		75 cm <small>Undyed/Incoloro</small>	U6094-B	U6093-B	U6092-B

Premium Reverse Cutting
Reverso Cortante Premium

1/2 Circle
Círculo ▼

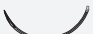
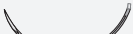
Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 0 EP 3.5
P48 48 mm		75 cm	P4891-75

Premium Reverse Cutting
Reverso Cortante Premium

3/8 Circle
Círculo ▼

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 6-0 EP 0.7	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3
PE11 11 mm		45 cm Undyed/Incoloro	PE1196-45B				
PE13 13 mm		45 cm Undyed/Incoloro	PE1396-45B	PE1395-45B	PE1394-45B		
		75 cm		PE1395-75			
PE16 16 mm		45 cm Undyed/Incoloro		PE1695-45B			
		75 cm Undyed/Incoloro		PE1695-75B	PE1694-75B		
PE19 19 mm		45 cm Undyed/Incoloro			PE1994-45B	PE1993-45B	
		75 cm Undyed/Incoloro			PE1994-75B	PE1993-75B	
PE24 24 mm		45 cm Undyed/Incoloro				PE2493-45B	
		75 cm				PE2493-75	
		75 cm Undyed/Incoloro				PE2493-75B	PE2492-75B

Taper Cutting
Redonda Punta Cortante

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP I EP 4	
W40 40 mm		90 cm	W4098-90	

Premium Taper Cutting
Redonda Punta Cortante Premium

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 3-0 EP 2	USP 2-0 EP 3
Y25 25 mm		70 cm	Y2593	Y2592

Premium Taper Cutting
Cuerpo Redondo Premium

J Needle
Aguja

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP I EP 4	
YJ30 30 mm		75 cm	YJ3098-75	

Premium Spatula
Espatulada Premium

1/4 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 6-0 EP 0.7	USP 5-0 EP I
EA6 6 mm		45 cm	EA696/2	
		45 cm	EA896/2	EA895/2
EA8 8 mm		45 cm Undyed/Incoloro	EA896/2-B	

Atramat® PGA

Premium Spatula Espatulada Premium

3/8 Circle Círculo


Thread Gauge Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 8-0 EP 0.4	USP 7-0 EP 0.5
EE6 6 mm		20 cm	EE6980-20	
		20 cm	EE6980/2-20	
		30 cm	EE6980/2-30	
EE6 6 mm		45 cm		EE697/2

Without Needle Sin Aguja

Thread Gauge Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP 1 EP 4	USP 2 EP 5
SS	Without Needle Sin Aguja	150 cm	SS0095-150	SS0094-150	SS0093-150	SS0092-150	SS0091-150	SS0098-150	SS0099-150
SS	Without Needle Sin Aguja	250 cm			SS0093-250	SS0092-250	SS0091-250	SS0098-250	SS0099-250
SS	Without Needle Sin Aguja	250 cm Reel/Carrete				SS0092-250C			
SS	Without Needle Sin Aguja	2 x 75 cm			SS0293-75	SS0292-75	SS0291-75	SS0298-75	
SS	Without Needle Sin Aguja	3 x 45 cm			SS0393-45	SS0392-45	SS0391-45		
SS	Without Needle Sin Aguja	5 x 70 cm				SS0592			SS0599
SS	Without Needle Sin Aguja	6 x 45 cm			SS0693-45		SS0691-45		
SS	Without Needle Sin Aguja	10 x 50 cm		SS1094-50	SS1093-50	SS1092-50			

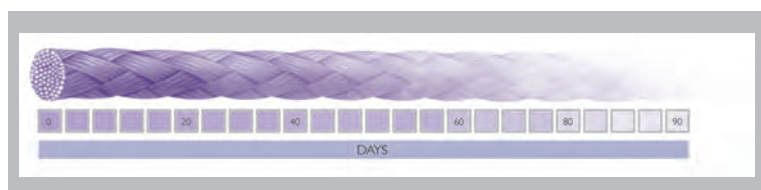Retains 70% tensile strength at 2nd week and 50% tensile strength at 3rd week

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Synthetic, absorbable, multifilament braided suture, in an undyed presentation.

Made of polyglycolic acid with polycaprolactone and calcium stearate coating.

The tissue reactivity in microscopic form is minimal.

Absorption occurs through progressive hydrolytic action, completed by 42 days.

The material retains approximately 50% of its tensile strength by the end of the first week.

Color code: Pink.

Frequently used in tissue that requires a short healing period.

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Braided structure = Excellent handling.

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Cylindrical coated multifilament thread = Smooth passage through the tissue.

Excellent knot coupling = Providing excellent security.

Gauge within U.S.P./E.P. guidelines = Less suture material implanted in the tissue.

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ABOUT

When implanted in the tissue, the ATRAMAT® Polyglycolic Acid Quick Absorption has minimal tissue reaction on a microscopic basis. This is comparable to other absorbable synthetic sutures of Polyglycolic Acid. However, ATRAMAT® Quick Absorption Polyglycolic Acid, offers a predictable absorption rate and high tensile strength over and above the USP standard, thus providing vital artificial support to the tissue during the critical healing period. The absorption occurs through progressive hydrolytic action completed in approximately 42 days. The material retains approximately 50% of its initial tensile strength at the end of the first week.

The ATRAMAT® Polyglycolic Acid Quick Absorption is coated with Polycaprolactone and Calcium Stearate in the following approximate percentages:

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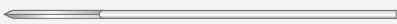
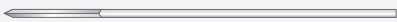
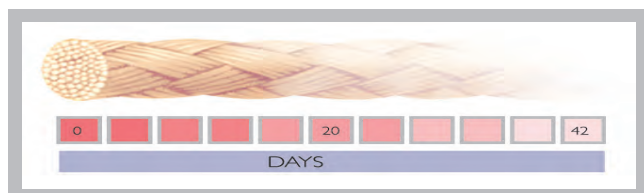
Retains 50% tensile strength at 1st week

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Made of a copolymer of glycolide acid and L-lactide [poly (glycolide-co-L-lactide)].

The tissue reactivity is minimal.

Absorption occurs through progressive hydrolytic action, completed between 56 and 70 days.

The material retains approximately 75% of its initial tensile strength by the end of the second week.

Color code: Intense Violet

Frequently used in soft tissue coaptation procedures.

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Cylindrical coated multifilament Thread = Smooth passage through the tissue.

Excellent knot coupling = Plasticity of the knot provides excellent security.

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ABOUT

When implanted in the tissue, the ATRAMAT® PGLA90 has minimal tissue reaction on a microscopic basis. However, ATRAMAT® PGLA90, offers a predictable absorption rate and high tensile strength above the USP standard, thus providing vital artificial support to the tissue during the critical healing period. The absorption occurs through progressive hydrolytic action completed in approximately 56 to 70 days. The material retains approximately 75% of its initial tensile strength at the end of the second week.

The suture has the following approximate percentages:

Glycolde 90%

L-lactide10%

Coating ≤1%

POLYGLACTIN 910 Poly(glycolide-co-lactide) (30/70) and calcium

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Synthetic, absorbable, multifilament braided suture, in undyed presentation.

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Color code: Red

Frequently used in approaching soft tissues, skin (intradermic) and mucuose, etc., that require a short healing period.

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ABOUT

The Atramat® PGLA90 Rapid POLYGLACTIN 910 Poly(glycolide-co-L-lactide) quick absorption surgical sutures, offers a predictable absorption rate and high tensile strength above the USP standard, thus providing vital artificial support to the tissue during the critical healing period. The absorption occurs through progressive hydrolytic action completed in approximately 42 days. The material retains approximately 50% of its tensile strength at the end of the first week.

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L-lactide 10%

Coating ≤1%

POLYGLACTIN 910 Poly(glycolide-co-lactide) (30/70) and calcium stearate

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Atramat® PGLA90 Rapid

Premium Regular Taper Point Ahusada Regular Premium

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 4-0 EP 1.5 USP 3-0 EP 2 USP 2-0 EP 3		
SR17 17 mm		75 cm Undyed/Incoloro	SR17294-75B		
SR26 26 mm		75 cm Undyed/Incoloro		SR26293-75B	SR26292-75B
SR30 26 mm		75 cm Undyed/Incoloro		SR30293-75B	SR30292-75B

Premium Heavy Taper Point Ahusada Gruesa Premium

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 2-0 EP 3 USP 0 EP 3.5 USP 1 EP 4		
		75 cm Undyed/Incoloro	SG36292-75B	SG36291-75B	
SG36 36 mm		90 cm Undyed/Incoloro		SG36291-90B	
		75 cm Undyed/Incoloro		SG40291-75B	SG40298-75B
SG40 40 mm		90 cm Undyed/Incoloro		SG40291-90B	

Premium Reverse Cutting Reverso Cortante Premium

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 5-0 EP 1 USP 4-0 EP 1.5 USP 3-0 EP 2		
PE13 13 mm		45 cm Undyed/Incoloro	PE13295-45B		
PE19 19 mm		45 cm Undyed/Incoloro		PE19294-45B	PE19293-45B
PE24 24 mm		75 cm Undyed/Incoloro			PE24293-75B


Atramat[®] PGLA90 Rapid

Without Needle
Sin Aguja

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 3-0 EP 2	USP 2-0 EP 3
SS	Without Needle Sin Aguja	150 cm Undyed/Incoloro	SS00293-150B	SS00292-150B

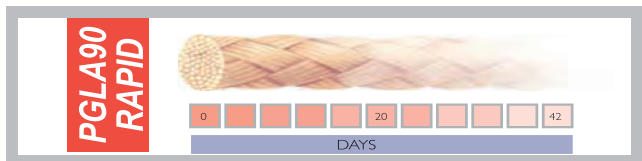Retains 50% Tensile strength at 1st week and completed absorption in 42 days aprox.

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Animal origin, absorbable suture with twisted multifilament, presented in brown.

Obtained from the small intestine serous layer of a healthy bovine (free of BSE and aphthous fever).

Because of its animal origin, the tissue reactivity is relatively moderate.

Absorbed by fagocytosis in approximately 60-90 days.

The thread keeps its tensile strength between 14 and 21 days. Specific patient conditions may cause a variation in the absorption period.

Color code: Brown

Frequently used in easy healing tissues and that do not require permanent artificial support.

CHARACTERISTICS + BENEFITS

High purity collagen (between 97% and 98%) = Minimum tissue reactivity.

Chromicizing process before twisting = Uniform chroming along the thread length, providing a reliable tissue support during absorption.

Uniform calibration and polishing monofilament appearance = Provides smooth passage through the tissue, minimizing tissue drag.

Sterilized by gamma rays = Complete and reliable sterilization, preserving the collagen natural characteristics.

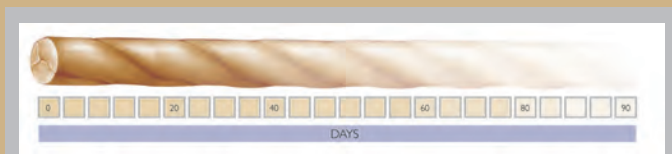
ABOUT

For: General, Gynecology, Obstetrics, Ophthalmic, Urology and other surgery.

Note: Caution must be taken when used in elderly, malnourished or immunologically deficient patients, in which the critical healing period of the wound may be delayed. Surgeon discretion is advised.

CONTRAINDICATIONS:

As a natural origin, absorbable suture, it should not be used where tissue support for an extended period of time is required.

Lost all tensile strength from 14 to 21 days aprox.

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Animal origin, absorbable, twisted multifilament suture presented in yellow.

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Absorbed by phagocytosis in approximately 50-70 days.

The thread keeps its tensile strength between 7 and 14 days. Specific patient conditions may cause the absorption period variation.

Color code: Yellow.

Frequently used in easy healing tissues that do not require permanent artificial support.

CHARACTERISTICS + BENEFITS

High purity collagen between 97% and 98% = Minimum tissue reactivity.

Uniform calibration and polishing monofilament appearance = Provides smooth passage through the tissue, minimizing drag.

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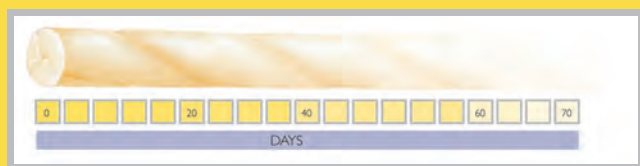
Note:

Caution must be taken when used in elderly, malnourished or immunologically deficient patients, in which the critical healing period of the wound may be delayed. In light of this, the surgeon may exercise with discretion.

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Loses all tensile strength from 7 to 14 days aprox.

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Regular Taper Point Ahusada Regular

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 5-0 EP 1	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP I EP 4
R20 20 mm		75 cm		R2024-75	R2023-75	R2022-75		
R26 26 mm		75 cm	R2625-75	R2624-75	R2623-75	R2622-75	R2621-75	R2628-75
R37 37 mm		75 cm		R3724-75	R3723-75	R3722-75	R3721-75	

Heavy Taper Point Ahusada Gruesa

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP I EP 4
G37 37 mm		75 cm	G3723-75	G3722-75	G3721-75	G3728-75
G40 40 mm		75 cm		G4022-75	G4021-75	G4028-75
G48 48 mm		90 cm		G4822-90		

Atramat® Plain Gut

Atramat® Catgut Simple

Reverse Cutting
Reverso Cortante

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type <i>Tipo</i>	Needle <i>Aguja</i>	Length <i>Longitud</i>	USP I EP 4
C48 48 mm		75 cm	C4818-75

Reverse Cutting
Reverso Cortante

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type <i>Tipo</i>	Needle <i>Aguja</i>	Length <i>Longitud</i>	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3
CE12 12 mm		45 cm	CE1224-45		
CE19 19 mm		75 cm	CE1924-75	CE1923-75	
CE24 24 mm		75 cm		CE2423-75	CE2422-75

Without Needle
Sin Aguja

Thread Gauge
Calibre de la Hebra

Type <i>Tipo</i>	Needle <i>Aguja</i>	Length <i>Longitud</i>	USP 4-0 EP 1.5	USP 3-0 EP 2	USP 2-0 EP 3	USP 0 EP 3.5	USP I EP 4
SS	Without Needle Sin Aguja	150 cm	SS0024-150	SS0023-150	SS0022-150	SS0021-150	SS0028-150

Synthetic, non-absorbable, monofilament suture, Presented in blue.

Extruded as a filament with uniform diameter controlled by a computer.

The tissue reaction is minimal.

The Polypropylene is extraordinarily stable; ideal for fulfilling its purpose for permanent tissue support without compromising its tensile strength.

Color code: Blue.

Frequently used to confront tissue in specialized areas. Plastic and Cardiovascular procedures among the most important ones.

CHARACTERISTICS + BENEFITS

Synthetic origin Inert Prime quality raw material = Ideal biocompatibility

Excellent plastodeformation = Adequate coupling of the knot providing excellent knot security.

Monofilament (uniform gauge) = Smooth passage through tissue, thus eliminating tissue drag.

Non-absorbable = Comprehensive and extended tissue support.

Resistance to repeated bending = Ideal for Cardiovascular procedures requiring support for the heart's propulsion, either in heart prosthesis or vascular anastomosis

ABOUT

Our non-traumatic needles, are precision sharpened and specially designed to provide a favourable cosmetic result. Furthermore, our needles have special bending (flexion) properties, resistance and hardness.

The Rockwell hardness of the needle assembled to Polypropylene is higher than the conventional needles. This allows excellent penetration and passage through calcified tissue or atheromatous plaques. ATRAMAT® has special needles for cardiovascular use, which provide easy penetration of the tissue, keeping a uniform diameter in the thread-to-needle ratio. This helps to avoid filtration, and is ideal for vascular use.

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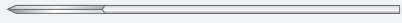
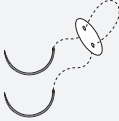
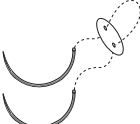
Synthetic, non-absorbable, multifilament braided suture, presented in a green or white.

Polyester made of Terephthalate with or without coating.

It's non-absorbable, synthetic origin means it has minimal tissue reaction.

Used in tissue coaptation due to it's characteristically high tensile strength.

Color code: Intense Orange

Frequently used in speciality surgery, including Cardiovascular and Ophthalmic procedures.

CHARACTERISTICS + BENEFITS

Synthetic origin. Inert. = Minimal tissue reaction.

Resistant to repeated flexion = Ideal for use in Cardiovascular Surgery (Heart Prosthesis)

Braided multifilament = Superior tensile strength.

Non-absorbable = Extended tissue support.

Green or white color = Good visibility in the surgical theatre.

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ABOUT

The Rockwell hardness of the needle assembled to the Polyester suture has a higher quality than the one used in conventional needles. This provides excellent penetration; easing the way through calcified tissue or atheromatous plaques. It is indicated in procedures where the use of a synthetic, braided, non-absorbable suture is recommended. These areas include: Cardiovascular, Orthopedic, Ophthalmic and General Surgery.

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Non-Absorbable Sutures

Atramat® Nylon

Monofilament, synthetic, non-absorbable suture, presented in black, blue or undyed.

Obtained from extrusion of polyamide (6.0 and 6.6) with a uniform cylindrical diameter.

Tissue reaction is minimal.

Nylon is a non-absorbable material that with time becomes encapsulated by connective tissue.

While the polyamide is non-absorbable, progressive hydrolysis in vivo may result in gradual loss of tensile strength over time.

Color code: Green

Normally used when confronting tissue in Ophthalmic and Plastic Surgery.

CHARACTERISTICS + BENEFITS

Synthetic origin (prime quality raw material). Inert = Low tissue reaction (upon implantation).

Excellent knot coupling = Providing excellent knot security.

Monofilament with uniform gauge = Smooth passage through tissue with little tissue drag.

Hermetically sealed packaging = Guaranteed product sterility.

Black, undyed and blue color = Excellent visibility in surgical field.

ABOUT

Its tensile strength and controlled elongation ensures the approximation of tissue borders. In addition, the sutures smooth passage through the tissue guarantees perfect healing. These physical attributes coupled with minimal tissue reaction, constitute an ideal suture for closure of a variety of tissues. Our ATRAMAT® Nylon materials are offered with a wide range of non-traumatic needles, precision sharpened, allowing for favorable cosmetic results.

NYLON (Blue, Black) For: Plastic, Gynecology, Obstetrics, Orthopedic and General surgery.

NYLON (Black) For: Ophthalmology, Micro-vascular surgery

NYLON (Undyed) For: Plastic and Reconstructive surgery

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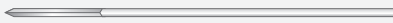
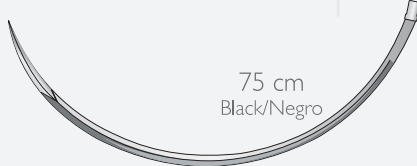
Conventional Cutting
Cortante Convencional

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 3-0 EP 2		USP 2-0 EP 3	
NE19 19 mm		45 cm Black/Negro	NE1943-N	NE1942-N		

Premium Spatula
Espatulada Premium

1/2 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 10-0 EP 0.2	
E6 6.2 mm		30 cm Black/Negro	E64100/2-30N	

Premium Spatula
Espatulada Premium

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 10-0 EP 0.2		USP 9-0 EP 0.3	
EE6 6.2 mm		30 cm Black/Negro	EE64100-30N			
EE6 6.2 mm		30 cm Black/Negro	EE64100/2-30N	EE6490/2-30N		
EE7 6.4 mm		30 cm Black/Negro	EE74100/2-30N	EE7490/2-30N		

Premium Taper Point
Ahusada Premium

3/8 Circle
Círculo

Thread Gauge
Calibre de la Hebra

Type Tipo	Needle Aguja	Length Longitud	USP 11-0 EP 0.1		USP 10-0 EP 0.2		USP 9-0 EP 0.3		USP 8-0 EP 0.4	
SE4 4 mm		13 cm Black/Negro	SE44110-13N							
		13 cm Black/Negro		SE54100-13N	SE5490-13N					
SE5 5 mm		8 cm Black/Negro		SE54100-8N	SE5490-8N					
SE6 6 mm		13 cm Black/Negro						SE6480-13N		

Non-Absorbable Sutures

Atramat® Silk

Natural, multifilament, braided, non-absorbable, silk suture, presented in black, white or blue.

Obtained from the cocoon of the Bombyx Mori silk worm.

Tissue reactivity may be moderate.

Tension is maintained over time; though it decreases while tissue capsulation occurs.

Color code: Light Blue.

Frequently used in tissue confrontation or ties except in Urological procedures.

CHARACTERISTICS + BENEFITS

Prime quality raw material = Prime quality guaranteed silk.

Threaded multifilament =The high technology used in the threading. guarantees a compact thread with a higher tensile strength.

Non-absorbable, natural origin = Longer tissue support.

Silicone Coating = Smooth passage through tissue.

Black, Blue, white color (virgin silk) = Versatile applications with / without silicone coating.

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ABOUT

Indicated in all surgical procedures provided the limitations of this material are known, like: General, Gastroenterology, Ophthalmology, Gynecology, and Obstretic Surgery.

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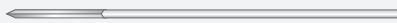
Non-Absorbable Sutures

Atramat® Stainless Steel

Monofilament, stainless steel, non-absorbable suture, presented in the material natural color.

The tissue reaction is minimal.

Permanent tensile strength.

Color code: Light Yellow.

Frequently used in Orthopedic Surgery including Sternum closure.

CHARACTERISTICS + BENEFITS

Stainless steel = Inert (compatible with tissue).

Non-absorbable = Extensive tissue support.

Monofilament = Smooth passage through tissue.

Permanent high tensile strength = Excellent support to tissue (bone structures).

Ductile and malleable = Good handling.

Special non-traumatic needles design = Excellent penetration in hard tissues.

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ABOUT

Once the users are aware of the characteristics, benefits and limitations of the stainless steel suture, and if they are familiar with good surgical practices, this product is adequate for procedures that require a highly resistant material for example, in Orthopedics and closure of the sternum.

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Reverse Cutting Spin Needle
Reverso Cortante Aguja Giratoria

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Atramat® Neoflex25

Partially Absorbable Surgical Mesh
Malla Parcialmente Absorbible

Our latest innovation in surgical meshes, the Atramat® Neoflex 25 surgical mesh composed of (Polypropylene + Poly(Glycolide-co-Epsilon-caprolactone)) has the unique feature of being partially absorbable through hydrolysis with in an established period of time, meanwhile it provides superior tissue support, less quantity of strange matter in the body, and a complete integration to the tissue, thus ensuring the bio-regeneration.

Also the “Tridirectional” elasticity provides the abdominal wall with natural movement while giving better comfort to the patient in recovery.

Neoflex 25 provides function, quality and security.

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Atramat® Propymesh

Non-Absorbable Surgical Mesh
Malla Quirúrgica No Absorbible

Our Polypropylene non-absorbable mesh Atramat® Propymesh is specifically designed to provide an outstanding surgical support with the plus of having “Tridirectional” elasticity, transparency and an extraordinary tensile force.

The unique design provides the adequate spaces for tissue integration favoring the anatomic movement of the abdominal wall.

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Aware of the several challenges during surgery, Atramat® provides the latest german technology in hemostatic clips made of the highest Titanium quality and a unique soft loading system, a special chevron shape for a safe “tips-first” closing action and a carefully serrated inner surface.

Our clips come in Large, Medium-Large, Medium, and Small sizes. They are available for open surgery or laparoscopic appliers.

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Atramat® Bonewax

Atramat® Cera para Huesos

The ATRAMAT® BONEWAX, is composed of a mixture of bee wax which is obtained by means of fusion with hot water in the walls of the honeycomb of the Apis mellifera Linne bee. It is obtained by purifying the bee wax and whitening the wax with Isopropyl Palmitate; a solvent exhibiting good emulsion properties in the

following proportions: bee wax 80% and Isopropyl Palmitate 20%.

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RAW MATERIAL:
Bee wax 80%,
Isopropyl Palmitate 20%

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COATING:
N/A

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PRESENTATION:
Box with 12, 24 and 36 pieces,
with individual pouches of 2.5 g.

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EXPIRY DATE:
5 years of manufacturing date

Our Plastic Surgery
Speciality Sutures

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AtraGlide®

Absorbable &
Non-Absorbable Threads

Our newest needle AtraGlide®, has the best design and technology, standing out through its performance.

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Needle tip with thin cutting edges = Easy and smooth passage through tissue.

New and innovative geometry = Greater penetration and controlled trajectory through tissue.

Special thermic coating = Allows for smoother passage through tissue with minimal tearing and tissue damage.

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www.atramat.com

México, 2013 | v.1.3 | Rev. 1

THE CONTENTS AND IMAGES SHOWN HERE ARE THE PROPERTY OF INTERNACIONAL FARMACÉUTICA, S.A. DE C.V., AND ARE DESIGNED TO BE USED EXCLUSIVELY FOR PROMOTIONAL AND DEMONSTRATIVE PURPOSES. UNDER NO CIRCUMSTANCES SHOULD THEY BE USED AS A MEDICAL REFERENCE. ALWAYS CONSULT A MEDICAL PROFESSIONAL PRIOR TO USING THIS OR ANY MEDICAL PRODUCT. INTERNACIONAL FARMACÉUTICA, S.A. DE C.V. PROVIDES THIS MATERIAL FOR THE EXCLUSIVE USE OF OUR DISTRIBUTORS AND OTHER PERSONS AUTHORIZED BY THE COMPANY IN ACCORDANCE WITH THE TERMS AND CONDITIONS ESTABLISHED BY THE COMPANY. INTERNACIONAL FARMACÉUTICA, S.A. DE C.V. CANNOT ACCEPT LIABILITY FOR DAMAGES CAUSED BY MISUSE OF ITS PRODUCTS OR OF THIS PROMOTIONAL MATERIAL.

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www.2hb.com.mx

We believe

“in all our people: employees, dealers, and allies; and in their will to do things better, pushing us and our products forward, that’s why we innovate...”

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Internacional Farmacéutica S.A. de C.V.